
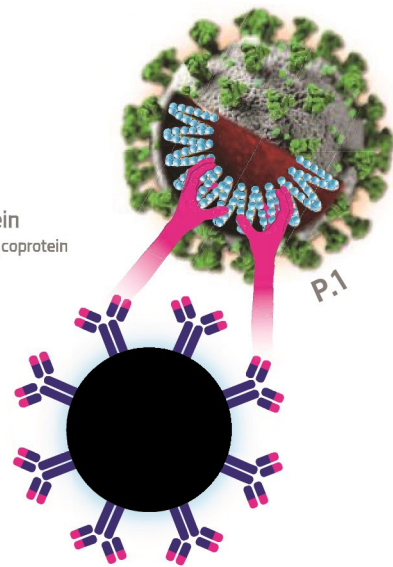
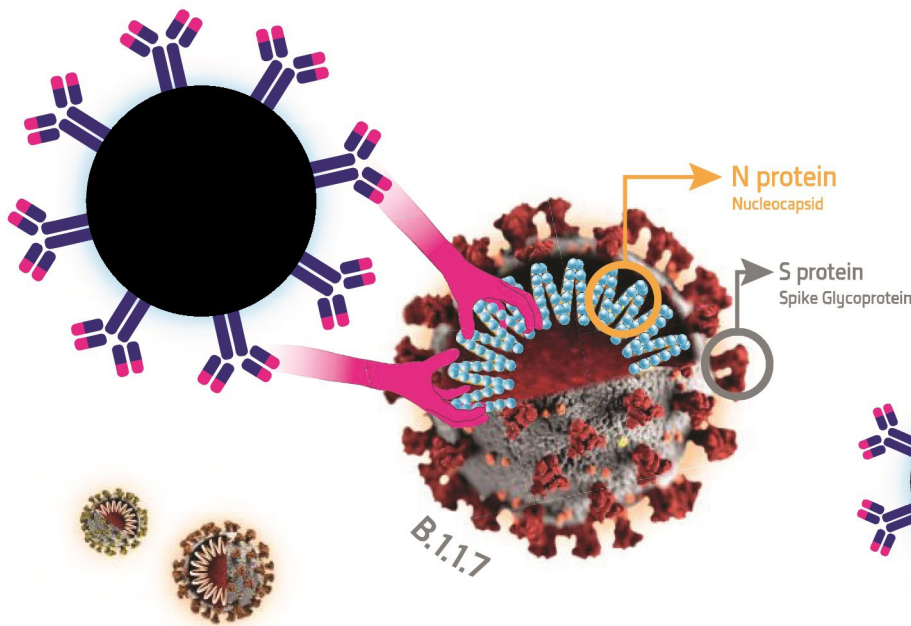



# Rapid COVID-19 Antigen Test Including Variants

One Step SARS-CoV-2 Antigen Rapid Test  
Time to Result 10~15 Minutes  
TGA Registered - ARTG# 365551

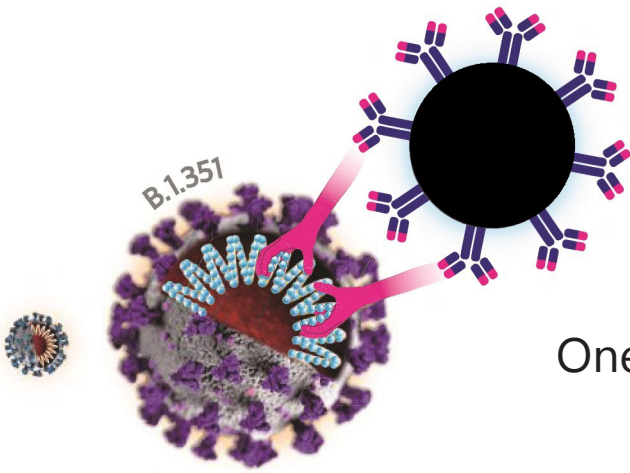

## DUAL COLOR SYSTEM

## Detection of New COVID-19 Variants

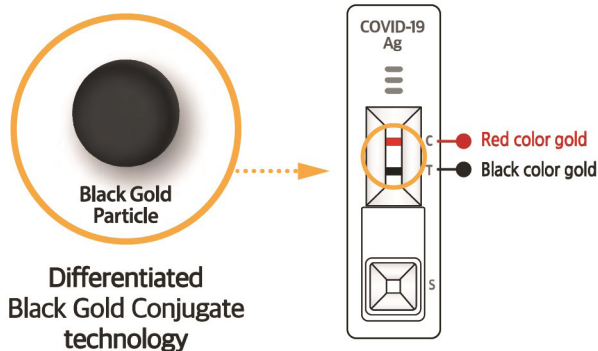

List of variants of SARS-CoV-2 Virus that BIOCREREDIT COVID-19 Ag can detect.

B.1.1.7	Alpha
B.1.351	Beta
B.1.429	Epsilon
P.1	Gamma
B.1.617	Delta
B.1.618	Indian Variant

Distributed By



Phone: +61 (0)3 9702 3766

Address: 16 Technology Circuit, Hallam, Victoria 3803, AUSTRALIA

Web: [www.haemokinesis.com](http://www.haemokinesis.com)

Email: [COVID19@haemokinesis.com](mailto:COVID19@haemokinesis.com)

# Detecting COVID-19 virus including multiple variants!

## BIOCREDIT COVID-19 Ag Products

### ■ Performance Characteristics

#### 【 Nasopharyngeal 】

- ▶ BIOCREDIT COVID-19 Ag has been evaluated with prospective clinical Nasopharyngeal specimens which were identified with the RT-PCR test in ITALY.

BIOCREDIT COVID-19 Ag	RT-PCR	
	Positive	Negative
Positive (+)	67	2
Negative (-)	5	209
<b>Total</b>	<b>72</b>	<b>211</b>
<b>Sensitivity</b>	<b>93.05% (95% CI: 83.86~97.41%)</b>	
<b>Specificity</b>	<b>99.05% (95% CI: 96.25~99.83%)</b>	

## Rapigen Offers

### 1. Quick Test

Result delivery in 10~15 minutes.

### 2. Easy Test

Point of Care (POCT) diagnostic test that requires **no equipment**.

### 3. Convenient Test

Everything required for test is within this kit.

### 4. TGA Test (ARTG# 365551)

Sensitivity (93.05%) and specificity (99.05%) - see above infograph.


### 5. COVID-19 Variant Test

Detecting COVID-19 virus including multiple variants including Delta (B.1.617), Alpha (B.1.1.7), Beta (B.1.351), Epsilon (B.1.429), Gamma (P.1), and Indian Variant (b=B.1.618).

### 6. Simple Reading Test

High visibility by adopting Dual Color System (Control line : Red, Test line : Black).

**No equipment required.**

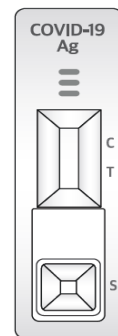

This product is not available to be purchased by, or supplied to, the general public. The product can be supplied to medical practitioners, health care professionals in residential and aged care facilities, accredited pathology laboratories, and federal, state and territory health departments. Refer to the Product Details and Limitations for important information in relation to the use of this product and results produced by using this product.